Clinician Headlight Stirnlampe für Mediziner Lámpara de cabeza para personal clínico Specchio di Clar Hoofdlamp voor artsen Foco Clínico

Clinician Headlight

Ideal for stoma examinations, the compact, lightweight design and comfortable fit make the Clinician Headlight well-suited for otolaryngologists, speech pathologists, audiologists as well as other medical staff.

HOW SUPPLIED

The Clinician Headlight is supplied with a headband and two AA batteries. Replacement bulb and headband are available separately.

REF	DESCRIPTION
BE9100	CLINICIAN HEADLIGHT
BE9150	CLINICIAN REPLACEMENT BULB
BE9160	CLINICIAN REPLACEMENT HEADBAND

PRODUCT SPECIFICATIONS

LAMP (BULB) TYPE	XENON
BATTERY BURN TIME	5-6 HOURS
LAMP (BULB) LIFE	20 HOURS
BATTERY TYPE	E91 1.5V ALKALINE AA-CELL (2)
BATTERY BRAND	EVEREADY ENERGIZER

ON/OFF INSTRUCTIONS

Have the light facing you (do not look directly into lamp). To turn light on, simply rotate the lens housing clockwise until light activates. **Do Not Overtighten.** To turn light off, turn the lens housing counterclockwise until light goes off. Make sure that when turning the light off, the lens is rotated a full turn off to prevent the light from accidentally coming on and discharging the batteries.

REPLACEMENT LAMP (BULB)

The Clinician Headlight comes with a plated parabolic reflector and prefocused subminiature Xenon lamp module which produces 800% more candlepower than a standard bulb. To change bulb:

1. Remove the lens and cover by turning counterclockwise.
2. Remove the lamp (bulb) module.
3. Replace the lamp (bulb) module. Replacement module BE9150 is available through InHealth Technologies.
4. Replace lens housing squarely on threads and turn clockwise while pressing on the lens and cover slightly.

BATTERY INFORMATION

The Clinician Headlight uses two AA batteries for power. To insert and change batteries:

1. Open body by unlatching side.
2. Remove used batteries and install fresh batteries, one positive (+) end up and one negative (-) end down.
3. Press both body and halves together until you hear a click.

Do not mix battery brands or use Nicaid or Zinc Carbon batteries or old/new batteries.

Caution: If light is not to be used for some time, remove batteries to prevent possible leakage and corrosion.

HEADBAND

The elastic fabric headband is adjustable and will fit to any size adult head. Replacement headband BE9160 is available through InHealth Technologies.

HEADLIGHT LIMITED WARRANTY

InHealth will repair or replace (at our option) the Clinician Headlight free of charge for 6 months from the date of purchase if the item is defective in workmanship or materials.

ORDERING INFORMATION

USA

InHealthCare products may be ordered directly from International HealthCare Technologies (InHealth) at the following toll free number: (800) 477-5969, Monday — Thursday, 9:30 am—7:00 pm; and Friday, 9:30 am—5:00 pm, Eastern Standard Time. The toll free number may be used from the continental USA, Alaska, Hawaii, Puerto Rico, and the Virgin Islands. Products also may be ordered at the InHealth website: www.inhealth.com

Canada

In Canada, please contact Auto Control Medical at (800) 461-0991, or Fax (800) 699-5936.

International

For a list of international distributors, please contact: International Healthcare Technologies
1110 Mark Avenue
Carpinteria, CA 93013-2918, USA
Tel: +01(805) 684-9337, Fax: +01(805) 684-8594

Consumer Affairs

If you have any questions or dissatisfaction with a product please contact customer service via telephone, fax, post, or E-mail. Tel: (800) 477-5969; Fax: (888) 371-1530; Post: 1110 Mark Avenue, Carpinteria, CA 93013-2918 USA; E-mail: productcomplaints@inhealth.com

RETURNED GOODS POLICY

All returned merchandise must have a Return Merchandise Authorization number (RMA), and must be unopened and undamaged. RMA numbers are obtained from InHealth Customer Service. Returns without an RMA will not be accepted. Products must be returned in unopened packages, with manufacturer's tamper evident seals intact to be accepted for replacement or credit. Products will not be accepted for replacement or credit if they have been in possession of the customer for more than 90 days. Special order products are nonreturnable. There is a 20% restocking fee on all returned merchandise.

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InHealth is a registered trademark of International Healthcare Technologies, A Division of Helix Medical, Inc. © 2004 International Healthcare Technologies / InHealth ist ein eingetragenes Warenzeichen von International HealthCare Technologies, ein Unternehmen der Helix Medical, Inc. / InHealth is a marca comercial registrada de International Healthcare Technologies, División de Helix Medical, Inc. / InHealth est une marque déposée de International HealthCare Technologies, filiale de Helix Medical, Inc. / InHealth é um marchio depositado di International Healthcare Technologies, una divisione di Helix Medical, Inc. / InHealth is een gedeponeerd handelsmerk van International Healthcare Technologies, een divisie van Helix Medical, Inc. / InHealth é uma marca registrada da International Healthcare Technologies, uma Divisão da Helix Medical, Inc.