# Blom-Singer®

voice restoration systems

## Blom-Singer® Voice Prosthesis Cleaning System

USER Instructions For Use

**INHEALTH**®  
TECHNOLOGIES

**R1**

37661-01B

37661-01B Effective October 2009 / 37661-01B Gültig ab Oktober 2009 /  
37661-01B Efectivo a partir de octubre de 2009 / 37661-01B Date d'entrée en vigueur :  
octobre 2009 / 37661-01B in vigore dall'ottobre 2009 / 37661-01B Met ingang van  
oktober 2009 / 37661-01B Vigente em outubro de 2009

Blom-Singer is a registered trademark of Hansa Medical Products, Inc.  
InHealth Technologies is a registered trademark of Helix Medical, LLC  
© 2009 InHealth Technologies, A Division of Helix Medical, LLC

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## **BLOM-SINGER® VOICE PROSTHESIS CLEANING SYSTEM**

### **Introduction**

*Please refer to the diagrams located at the back of this instruction manual. A glossary of terms appears at the end of this chapter.*

The Voice Prosthesis Cleaning System accessories are an integral part of daily maintenance for tracheoesophageal voice prostheses. The cleaning system consists of a Cleaning Brush (diagram 1) and Flushing Device (diagram 2).

The voice prosthesis and accessory items should only be used by individuals who have received proper training in the effective management and care of Blom-Singer voice prostheses. Management includes, but is not limited to, handling of the device, dislodgement of the device, and cleaning of the device.

Your doctor will discuss with you the risks and benefits of using a voice prosthesis.

The information in this manual is not intended to be used in place of consultation or instructions of a medical professional.

**This product is intended for use by a single patient only.** See Warnings, Precautions and Complications.

### **INDICATIONS**

The Voice Prosthesis Cleaning System is designed for use only with Blom-Singer voice prostheses. The cleaning brush and flushing device provide a means for careful cleaning of the voice prosthesis while it is in the patient's tracheoesophageal puncture.

### **CONTRAINDICATIONS**

The Voice Prosthesis Cleaning System are medical products that should be used only by individuals with experience and training in their use and care.

### **HOW SUPPLIED**

The cleaning brush and flushing device are available nonsterile separately as a 3-pack. The following chart depicts which cleaning device is used with the appropriate voice prosthesis.

Blom-Singer® Indwelling Voice Protheses	Flushing Device	Cleaning Brush
CLASSIC, 16 & 20 Fr.	•	•
ADVANTAGE® (Hard Valve Assembly) 20 Fr.	•	•
ADVANTAGE® (Soft Valve Assembly) 16 & 20 Fr.	•	•
DUAL-VALVE™, 20 Fr.	•	•
Blom-Singer® Voice Protheses (patient changeable)	Flushing Device	
Low Pressure Voice Prosthesis, 16 & 20 Fr.	•	
Duckbill Voice Prosthesis, 16 Fr.	•	

## WARNINGS AND PRECAUTIONS

InHealth relies on the qualified, trained medical professional to instruct patients on the use and care of this device and to provide patients with the Blom-Singer Voice Prosthesis Cleaning System instructions for use supplied with the product.

If there are tears, cracks, or structural damage to the voice prosthesis or cleaning devices, discontinue use.

### **Voice Production**

The voice prosthesis must be kept clean of mucus and organic matter in order for it to function properly and to help you speak with a good quality of voice.

### **Voice Prosthesis Dislodgement**

If the voice prosthesis is accidentally dislodged from the TE puncture, immediately place a Blom-Singer puncture dilator, or a suitable device as recommended by your doctor, of the appropriate diameter, in the puncture to keep it from closing and leaking fluids. Contact your doctor immediately. The voice prosthesis should be reinserted within 24 hours.

Care should be exercised when inserting or removing cleaning devices to avoid accidental displacement of the prosthesis which

---

could result in aspiration (inhalation) of the voice prosthesis. In the unlikely event that this should occur, you should attempt to cough the voice prosthesis out of the windpipe. Immediately seek medical attention if unsuccessful.

Never insert foreign objects into the voice prosthesis. The cleaning device is designed for only cleaning dried mucus and organic matter from the voice prosthesis. Inserting objects other than the supplied cleaning device may cause dislodgment of the voice prosthesis or its components and may cause these objects to be swallowed or inhaled.

### **Voice Prosthesis Leakage**

Always use gentle handling and pressure when cleaning the voice prosthesis to avoid deterioration of the device and damaging internal components which could cause leakage.

Recurrent leakage of fluids through the voice prosthesis should be evaluated by your clinician soon after it is noticed.

### **Candida species**

Most users have a yeast (*Candida albicans*) in their mouth, throat, or esophagus; this is a commonly occurring condition. However, if the prosthesis is not properly cleaned in users who have this condition, the yeast may grow on the valve of the voice prosthesis. The yeast is identified by discolored spots on the prosthesis. Yeast deposits can cause valve deformation and failure, i.e., liquid leakage through the prosthesis and an increase in the pressure necessary to voice. Replacement of the prosthesis may be required more frequently. Some patients can be treated successfully for yeast with oral medication or mouth rinses.

### **Device Removal (Indwelling Voice Prostheses)**

Never attempt to remove the device by yourself or allow anyone other than a qualified, trained medical professional to remove the voice prosthesis.

**Caution:** Federal (USA) law restricts this device to sale by or on the order of a physician or a licensed speech pathologist.

## **COMPLICATIONS**

Refer to the voice prosthesis instructions for use for related complications.

## VOICE PROSTHESIS CLEANING AND CARE INSTRUCTIONS

*Please refer to the diagrams located at the back of this instruction manual.*

Your clinician will instruct you on the routine care and cleaning of the Blom-Singer voice prosthesis.

The voice prosthesis will continue to function and may last longer if it is cleaned daily with the supplied cleaning device per the recommended instructions.

The cleaning device provides a means for careful cleaning of the voice prosthesis while in your TE puncture. The cleaning device is designed for use only in cleaning organic matter from the voice prosthesis.

**Always** use **gentle** handling and pressure when cleaning the voice prosthesis to avoid deterioration of the device and damaging internal components which could cause leakage.

**Caution:** Use **only** supplied cleaning device. **Do not** insert foreign objects into voice prosthesis. Inserting objects other than the supplied cleaning device may cause dislodgment of the voice prosthesis or its components.

**Warning:** Care should be exercised when inserting or removing cleaning devices to avoid accidental displacement of the prosthesis which could result in aspiration (inhalation) of the voice prosthesis. In the unlikely event that this should occur, you should attempt to cough the voice prosthesis out of the windpipe. Immediately seek medical attention if unsuccessful.

The cleaning device and user's hands **should always be thoroughly** clean to prevent contaminants from entering into the tracheostoma, puncture, or esophagus.

Cleaning of the voice prosthesis should only be done in front of a mirror with a bright light focused directly on the stoma so that the open end of the prosthesis is clearly visible.

Before use, the cleaning device should be rinsed with 3% hydrogen peroxide. Use long handled tweezers to carefully remove any dried phlegm that may be in the open end of the voice prosthesis.

---

Never use facial or toilet tissue to dry a cleaning device as particles of lint or fabric may be deposited onto the device that can be inhaled into the airway upon reinsertion.

Do not use solvents or petroleum-based products for cleaning or lubricating the device. These materials may damage silicone or cause the device to not work properly.

Repeat cleaning regimen as needed. The cleaning device should be cleaned, rinsed and air-dried after each use. Place the cleaning device in 3% hydrogen peroxide after each use. Shake to remove any residual liquid. Allow the cleaning device to dry in a well-lit, aerated area. If desired, these accessories can be washed in soap and water, rinsed and air-dried.

### **Cleaning Brush**

Prior to using the cleaning brush, inspect it for any structural damage; do not use a device that is damaged. Moisten the cleaning brush in clean water before use. **Carefully** and **gently** insert the cleaning brush fully into the indwelling voice prosthesis (diagram 3). **Never** force the cleaning brush through the voice prosthesis. **Gently** rotate the cleaning brush clockwise and counterclockwise multiple times to clean dried phlegm from the inside of the prosthesis body and the valve. If the cleaning brush will not readily rotate, the voice prosthesis may be plugged and should therefore be flushed with the flushing device to loosen any dried phlegm (refer to Flushing Device instructions). After brushing, remove the cleaning brush carefully.

### **Flushing Device**

To fill the flushing device with water, squeeze the bulb and then release it in clean water. Shake excess water from the tip of the flushing device before flushing the voice prosthesis. **Carefully** and **gently** insert the tip of the flushing device into the voice prosthesis until it fits completely against the opening of the prosthesis (diagram 4). (**Note:** Do **not** flush water through the voice prosthesis until the tip of the flushing device is properly inserted into the prosthesis, thus avoiding leaking water into the trachea. Should a small amount of water leak into the trachea, this will cause coughing.) Squeeze the bulb on the flushing device to flush a rapid squirt of water through the voice prosthesis. After

flushing, remove the flushing device carefully. If water will not readily squirt through the voice prosthesis, this indicates that it may be plugged with dried phlegm. Allow a few drops of water to dissolve the dried matter for a few minutes and then flush again with the flushing device until the matter is loosened.

## **ORDERING INFORMATION**

### **USA**

Blom-Singer products may be ordered directly from InHealth Technologies. TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337, Monday — Friday, 9:30 am — 7:00 pm, Eastern Standard Time. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: [order@inhealth.com](mailto:order@inhealth.com) ORDER ON-LINE: [www.inhealth.com](http://www.inhealth.com) POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service.

### **International**

Please contact our customer service department for a distributor referral.

### **Consumer Affairs**

If you have any questions or dissatisfaction with a product please contact our customer service department via telephone, fax, post, or Email: [productcomplaints@inhealth.com](mailto:productcomplaints@inhealth.com)

## **RETURNED GOODS POLICY**

All returned merchandise must have a Return Merchandise Authorization number (RMA), and must be unopened and undamaged. RMA numbers are obtained from InHealth Customer Service. Returns without an RMA will not be accepted. Products must be returned in unopened packages, with manufacturer's tamper evident seals intact to be accepted for replacement or credit. Products will not be accepted for replacement or credit if they have been in possession of the customer for more than 14 days. Special order products are nonreturnable. There is a 20% restocking fee on all returned merchandise.

## **LIMITED WARRANTY, LIMITED REMEDY AND LIMITATION OF LIABILITY**

InHealth Technologies, a Division of Helix Medical, LLC, warrants that the Product will be free from defects in workmanship

and materials on the date of purchase. THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES OR REPRESENTATIONS, EXPRESS OR IMPLIED, ARISING BY LAW OR CUSTOM, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SAID IMPLIED WARRANTIES BEING HEREBY EXPRESSLY DISCLAIMED. Purchaser's sole and exclusive remedy for a valid warranty claim is replacement of the Product. This remedy does not include the cost of installation, removal, dismantling, delivery or reinstallation. InHealth shall have no liability to the extent Products are or have been modified or subject to misuse, abuse or improper storage, installation or maintenance or damaged in transit. No returns will be accepted without prior approval and authorization. INHEALTH SHALL NOT BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT, CONSEQUENTIAL, PUNITIVE, OR EXEMPLARY DAMAGES OF ANY KIND.

## **GLOSSARY**

### **Candida albicans**

A yeast fungus that is a commonly occurring condition that can infect the mouth or the skin and may multiply and overgrow on to the voice prosthesis.

### **Clinician**

A health professional, such as a physician, licensed speech-language pathologist, or nurse who is involved in clinical practice.

### **Contraindications**

A reason that makes it inadvisable to prescribe a particular procedure or treatment.

### **Esophagus**

The muscular membranous tube for the passage of food from the pharynx to the stomach; the "food pipe."

### **Fistula; TE-fistula**

An abnormal duct or passage that connects an abscess or cavity to the body surface; a small opening between the trachea and the esophagus.

### **Indications**

A reason to prescribe a drug or perform a procedure.

### **Tracheoesophageal puncture (TEP)**

An opening made by a surgeon between the trachea and the esophagus. The opening is made at the time of initial surgery or later so that a voice prosthesis can be placed into this passage.

### **Trachea**

The airway through which respiratory air passes to and from the lungs (windpipe).

### **Tracheoesophageal (TE)**

Pertaining to the trachea and esophagus.

### **Tracheostoma**

An opening through the neck and into the trachea. It is through this opening that the laryngectomee breathes.

### **Voice prosthesis**

A one-way speech valve that is inserted into the TEP to allow air to pass from the trachea through the prosthesis and into the esophagus to produce sound.

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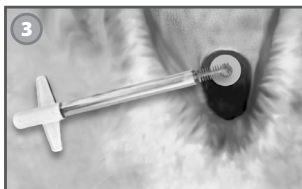
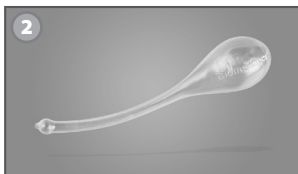
---

## BIBLIOGRAPHY / BIBLIOGRAFIE / BIBLIOGRAFÍA / BIBLIOGRAFIA

Additional references available upon request.

Weitere Referenzen aus der Literatur sind auf Anfrage erhältlich. / Se pueden solicitar más referencias. / Autres références disponibles sur demande. / Riferimenti aggiuntivi sono disponibili a richiesta. / Meer informatie is op aanvraag beschikbaar. / Referências adicionais mediante solicitação.

1. Blom, E.D.: "Some Comments on the Escalation of Tracheoesophageal Voice Prosthesis Dimensions," **Arch Otolaryngol**, 129: 500-502, 2003.
2. Eerenstein, S.E., Grolman, W., Schouwenburg, P.F.: "Downsizing of Voice Prosthesis Diameter in Patients with Laryngectomy," **Arch Otolaryngol**, 128: 838-841, 2003.
3. Hamaker, R.C., Blom, E.D.: "Botulinum Neurotoxin for Pharyngeal Constrictor Muscle Spasm in Tracheoesophageal Voice Restoration," **Laryngoscope**, 113:1479-1482, 2003.
4. Evans, P.H.R., Blom, E.D.: "Functional Restoration of Speech," **Principles and Practice of Head and Neck Oncology**, Evans, P.H.R., Montgomery, P.W., Gullane, P.J. (Eds.) Taylor and Francis Group, London, 2003.
5. Blom, E.D., Singer M.I., and Hamaker, R.C., **Tracheoesophageal Voice Restoration Following Total Laryngectomy**. San Diego: Singular Publishing Group, Inc.1998.
6. Blom, E.D., Hamaker, R.C.: "Tracheoesophageal Voice Restoration Following Total Laryngectomy," **Cancer of the Head and Neck**, E.N. and Suen, J. (Eds). W.B. Saunders Publishers. 839-852, 1996.
7. Blom, E.D., "Tracheoesophageal Speech," **Seminars in Speech and Language**. New York: Thieme Medical Publishers. 16:3, 191-204, 1995.
8. Pauloski, B.R., Blom, E.D., Logemann, J.A., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part II: Swallow Characteristics," **Laryngoscope**, 105:10: 1104-1110, 1995.
9. Blom, E.D., Pauloski, B.R., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part I: Speech Characteristics," **Laryngoscope**, 105:10: 1093-1103, 1995.

- 
10. Blom, E.D., Hamaker, R.C., Freeman, S.B.: "Postlaryngectomy Voice Restoration," Lucente, F.E. (Ed.): **Highlights of the Instructional Courses**, Vol. 7. St. Louis: Mosby-Year Book. pp 3-10, 1994.
  11. Blom, E.D., Singer M.I., and Hamaker, R.C. "A Prospective Study of Tracheoesophageal Speech," **Arch Otolaryngol**, 112: 440-447, 1986.
  12. Blom, E.D., Singer, M.I. "Disinfection of Silicone Voice Prostheses," **Arch Otolaryngol**, 112: 1986.
  13. Blom, E.D., Singer M.I., and Hamaker, R.C. "An Improved Esophageal Insufflation Test," **Arch Otolaryngol**, 111: 211-212, 1985.
  14. Hamaker, R.C., Singer M.I., Blom E.D. and Daniels, H.A. "Primary Voice Restoration at Laryngectomy," **Arch Otolaryngol**, 111: 182-186, 1985.
  15. Singer, M.I. and Blom, E.D. "A Selective Myotomy for Voice Restoration After Total Laryngectomy," **Arch Otolaryngol**, 107: 670-673, 1981.
  16. Singer, M.I. and Blom, E.D. "An Endoscopic Technique for Restoration of Voice After Laryngectomy," **Ann Otol Rhinol Laryngol**, 89:6: 529-533, 1980.

**INHEALTH**<sup>®</sup>  
T E C H N O L O G I E S

---

CE

**InHealth Technologies**

A Division of Helix Medical, LLC  
1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
[www.inhealth.com](http://www.inhealth.com)

**EMERGO EUROPE**

Molenstraat 15  
2513 BH, The Hague  
The Netherlands