# INHEALTH™

TECHNOLOGIES

## Blom-Singer® Indwelling Voice Prostheses

USER Instructions For Use

**Blom-Singer®**

voice restoration systems

**R1**

37639-01F

37639-01F Effective January 2016 / Gültig ab Januar 2016 /  
Efectivo a partir de enero de 2016 / Date d'entrée en vigueur :  
janvier 2016 / Valido da gennaio 2016 / Met ingang van januari  
2016 / Vigente em janeiro de 2016 / V platnosti od ledna 2016 /  
Wchodzi w życie w styczniu 2016 r.

**Blom-Singer** and **InHealth Technologies** are registered  
trademarks of **Freudenberg Medical, LLC**

Patent(s): [www.inhealthpatents.com](http://www.inhealthpatents.com)

# TABLE OF CONTENTS

Inhalt / Índice / Sommaire / Sommario / Inhoud / Obsah / Spis treści

<b>ENG</b>	<b>Indwelling Voice Protheses</b>	<b>4</b>
<b>DEU</b>	<b>Verweil-Stimmprothesen</b>	<b>13</b>
<b>SPA</b>	<b>Prótesis fonatorias fijas</b>	<b>23</b>
<b>FRA</b>	<b>Prothèses vocales à demeure</b>	<b>33</b>
<b>ITA</b>	<b>Protesi fonatoria fissa</b>	<b>42</b>
<b>NLD</b>	<b>Verblijfstemprothese</b>	<b>51</b>
<b>POR</b>	<b>Prótese vocal fixa</b>	<b>60</b>
<b>CES</b>	<b>Vnitřní hlasové protézy</b>	<b>70</b>
<b>POL</b>	<b>Stałe protezy głosowe</b>	<b>79</b>

## **Blom-Singer®**

voice restoration systems

### **USER Instructions For Use**

Gebrauchsanweisung für BENUTZER  
Instrucciones para el USUARIO  
Instructions d'utilisation destinées à L'UTILISATEUR  
Istruzioni per l'uso destinate ALL'UTENTE  
Gebruiksaanwijzing voor de GEBRUIKER  
Instruções de uso para o USUÁRIO  
Návod k použití pro UŽIVATELE  
Instrukcja użytkowania dla UŻYTKOWNIKA

## **BLOM-SINGER® INDWELLING VOICE PROSTHESES**

### **Introduction**

*Please refer to the diagrams located at the back of this instruction manual. A glossary of terms appears at the end of this chapter.*

The Blom-Singer voice prosthesis has helped thousands of laryngectomees worldwide to speak again. The original tracheoesophageal voice prosthesis was developed by Dr. Eric Blom and Dr. Mark Singer in 1978. Today, their method of TEP, tracheoesophageal puncture, is still recognized throughout the world as the leading standard for voice restoration following total laryngectomy.

With persistence, a laryngectomee can learn from a clinician or speech therapist new ways of speaking. One option, described in this manual, enables voice restoration using a tracheoesophageal voice prosthesis (diagram 1). The voice prosthesis (1A) is inserted into a TE puncture by your clinician which allows air to enter from the lungs into the trachea (1B) and esophagus (1C) to produce sound (1D).

The voice prosthesis and accessory items should only be used by individuals who have received proper training in the effective management and care of Blom-Singer voice prostheses. Management includes, but is not limited to, handling of the device, dislodgement of the device, and cleaning of the device.

Your doctor will discuss with you the risks and benefits of using a voice prosthesis. The information in this manual is not intended to be used in place of consultation or instructions of a medical professional.

**This Blom-Singer medical device may be used only once by one patient. It may not be reused.** See Warnings, Precautions and Complications.

### **INDICATIONS**

The Blom-Singer Indwelling voice prosthesis is indicated for tracheoesophageal voice restoration following total laryngectomy, when placement, or replacement, of an Indwelling voice prosthesis is performed by a qualified, trained medical professional.

## CONTRAINDICATIONS

The Blom-Singer Indwelling prosthesis is a medical product that should be used only by a qualified, medical professional with experience and training in its use and care.

## HOW SUPPLIED

The Blom-Singer Indwelling voice prosthesis is supplied as a complete voice restoration system for primary, secondary, or replacement procedures. Included in each package is a Voice Prosthesis Cleaning System for the patient. Accessory products for each prosthesis are available separately.

## PRODUCT DESCRIPTION

Four distinct styles and the largest selection of sizes are available to allow your clinician to choose a voice prosthesis best suited for your needs.

The Blom-Singer voice prosthesis is made of silicone, a biocompatible material used in many medical products. The gel cap used for insertion of the prosthesis is made of a vegetable base gelatin. Consult with your physician as individual reactions to these materials may occur.

The Blom-Singer voice prosthesis, (diagram 2), consists of a one-way silicone flap valve (diagram 2a), an esophageal retention flange (diagram 2b), a body that holds the valve assembly (diagram 2c), a tracheal retention flange (diagram 2d), and a neck strap (diagram 2e).

Blom-Singer® Indwelling Voice Protheses	Non-sterile	Sterile
CLASSIC, 16 & 20 Fr.	•	•
ADVANTAGE® (Hard Valve Assembly) 20 Fr.	•	
ADVANTAGE® (Soft Valve Assembly) 16 & 20 Fr.	•	
DUAL-VALVE™, 20 Fr.	•	

---

## **BLOM-SINGER ACCESSORIES**

Blom-Singer accessories are available separately; please refer to the InHealth website, [www.inhealth.com](http://www.inhealth.com), or contact customer service for product details.

### **Voice Prosthesis Cleaning System**

The Voice Prosthesis Cleaning System contains the necessary cleaning devices for careful cleaning of the voice prosthesis while in the TE puncture.

### **Puncture Dilator**

The Blom-Singer Puncture Dilator is used in an emergency situation to prevent the TE puncture from closing when an Indwelling voice prosthesis has been accidentally dislodged, or when it is removed by your clinician. It is recommended that you always have on hand a puncture dilator, of the appropriate diameter, or a suitable device recommended by your doctor that serves the same function. The Blom-Singer Puncture Dilator should only be used by individuals with experience and training in the use and care of TEP prosthetic devices.

### **Valved and Plug Inserts**

The valved and plug inserts are designed to temporarily stop the voice prosthesis from leaking until professional, medical attention can be obtained. The valved insert allows the user to continue voicing but the plug insert does not allow voicing.

## **WARNINGS AND PRECAUTIONS**

InHealth relies on the qualified, trained medical professional to instruct patients on the use and care of this device and to provide patients with the Blom-Singer Indwelling Voice Prostheses instructions for use supplied with the product.

Should the voice prosthesis show signs of malfunction, contact your physician/clinician as soon as possible to have the device performance evaluated.

If there are tears, cracks, or structural damage to the voice prosthesis or cleaning devices, discontinue use.

Changes in your anatomy or medical status may lead to improper fitting and/or function of the device. It is recommended that your voice prosthesis and TEP be evaluated by your clinician at regular intervals.

### **Voice Production**

To prevent post-operative complications, do not begin speaking with the voice prosthesis until your doctor has informed you that it is safe for you to do so.

The voice prosthesis must be kept clean of mucus and organic matter in order for it to function properly and to help you speak with a good quality of voice.

### **Voice Prosthesis Dislodgement**

If the voice prosthesis is accidentally dislodged from the TE puncture, immediately place a Blom-Singer Puncture Dilator, or a suitable device as recommended by your doctor, of the appropriate diameter, in the puncture to keep it from closing and leaking fluids. Contact your doctor immediately. The voice prosthesis should be reinserted within 24 hours.

Care should be exercised when inserting or removing cleaning devices to avoid accidental displacement of the prosthesis which could result in aspiration (inhalation) of the voice prosthesis. In the unlikely event that this should occur, you should attempt to cough the voice prosthesis out of the windpipe. Immediately seek medical attention if unsuccessful.

Never insert foreign objects into the voice prosthesis. The cleaning device is designed for only cleaning dried mucus and organic matter from the voice prosthesis. Inserting objects other than the supplied cleaning device may cause dislodgment of the voice prosthesis or its components and may cause these objects to be swallowed or inhaled.

### **Voice Prosthesis Leakage**

Always use gentle handling and pressure when cleaning the voice prosthesis to avoid deterioration of the device and damaging internal components which could cause leakage.

Recurrent leakage of fluids through the voice prosthesis should be evaluated by your clinician soon after it is noticed. When the flap valve (diagram 2a) fails to close completely a few drops of fluid may pass back through the valve from the esophagus to the trachea, which may cause coughing.

---

### **Candida species**

Most users have a yeast (*Candida albicans*) in their mouth, throat, or esophagus; this is a commonly occurring condition. However, if the prosthesis is not properly cleaned in users who have this condition, the yeast may grow on the valve of the voice prosthesis. The yeast is identified by discolored spots on the prosthesis. Yeast deposits can cause valve deformation and failure, i.e., liquid leakage through the prosthesis and an increase in the pressure necessary to voice. Replacement of the prosthesis may be required more frequently. Some patients can be treated successfully for yeast with oral medication or mouth rinses.

### **Device Removal**

Never attempt to remove the device by yourself or allow anyone other than a qualified, trained medical professional to remove the voice prosthesis.

The Indwelling voice prosthesis is not a permanent device and requires replacement periodically by your doctor.

The Indwelling voice prosthesis may be left in place in the TE puncture until it ceases to function correctly, that is, until it leaks or is not providing adequate voice for speech. Many clinicians recommend routine follow-up evaluations not longer than 6 months.

**Caution:** Federal (USA) law restricts this device to sale by or on the order of a physician or a licensed speech pathologist.

### **COMPLICATIONS**

Although rare, the following complications have been identified to occur with silicone prostheses of the Blom-Singer type. They include:

- stoma contamination or bacterial infection, which may require removal of the prosthesis and/or appropriate antibiotics;
- accidental inhalation of the voice prosthesis into the airway, which may require removal by a physician;
- inhalation of liquids through the prosthesis into the airway, which may cause the patient to cough;
- occasional movement out of position by the voice prosthesis, requiring replacement after enlargement of the TE puncture

- and additional supervision of the stoma care regimen;
- puncture enlargement resulting in leakage of fluids around the prosthesis;
- inflammatory reaction around the puncture site and formation of scar tissue;
- dislodgment of the voice prosthesis and subsequent closure of the TE puncture;
- chronic and persistent leakage around the voice prosthesis, requiring surgical revision or closure of the puncture;
- difficulty speaking or swallowing;
- tearing or other damage to the voice prosthesis from improper use;
- yeast deposits causing leakage through the prosthesis or inability of the valve to function properly;
- accidental swallowing of the prosthesis into the esophagus and/or gastrointestinal (GI) tract.

## **VOICE PROSTHESIS CLEANING AND CARE INSTRUCTIONS**

Your clinician will instruct you on the routine care and cleaning of the Blom-Singer Indwelling voice prosthesis.

The following information is a brief overview of the proper care and maintenance of the voice prosthesis. Please refer to the Voice Prosthesis Cleaning System instructions for use supplied with this product for complete instructions on how to use the cleaning devices.

The voice prosthesis will continue to function and may last longer if it is cleaned daily with the supplied cleaning device per the recommended instructions.

The cleaning device provides a means for careful cleaning of the voice prosthesis while in your TE puncture. The cleaning device is designed for use only in cleaning organic matter from the voice prosthesis.

**Always** use **gentle** handling and pressure when cleaning the voice prosthesis to avoid deterioration of the device and damaging internal components which could cause leakage.

---

**Caution:** Use **only** supplied cleaning device. **Do not** insert foreign objects into voice prosthesis. Inserting objects other than the supplied cleaning device may cause dislodgment of the voice prosthesis or its components.

The cleaning device and user's hands should **always be thoroughly clean** to prevent contaminants from entering into the tracheostoma, puncture, or esophagus. Cleaning of the Indwelling voice prosthesis should only be done in front of a mirror with a bright light focused directly on the stoma so that the open end of the prosthesis is clearly visible.

Never use facial or toilet tissue to dry a cleaning device as particles of lint or fabric may be deposited onto the device that can be inhaled into the airway upon reinsertion.

Do not use solvents or petroleum-based products for cleaning or lubricating the device. These materials may damage silicone or cause the device to not work properly.

## **SPECIAL ORDER PRODUCTS**

If this instruction manual accompanies a Special Order Product, there may be differences in the physical characteristics between the product and the product descriptions described in this instruction manual. These differences will not affect the safety or desired beneficial effect of the special order product. Special Order products are nonreturnable.

## **ORDERING INFORMATION**

### **USA**

Blom-Singer products may be ordered directly from InHealth Technologies. TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337, Monday — Friday, 9:30 am — 7:00 pm, Eastern Standard Time. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: [order@inhealth.com](mailto:order@inhealth.com) ORDER ON-LINE: [www.inhealth.com](http://www.inhealth.com) POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service.

### **International**

Please contact our customer service department for a distributor referral.

## **Consumer Affairs**

If you have any questions or dissatisfaction with a product please contact our customer service department via telephone, fax, post, or Email: [productcomplaints@inhealth.com](mailto:productcomplaints@inhealth.com)

## **GLOSSARY**

### **Candida albicans**

A yeast fungus that is a commonly occurring condition that can infect the mouth or the skin and may multiply and overgrow on to the voice prosthesis.

### **Clinician**

A health professional, such as a physician, licensed speech-language pathologist, or nurse who is involved in clinical practice.

### **Contraindications**

A reason that makes it inadvisable to prescribe a particular procedure or treatment.

### **Esophagus**

The muscular membranous tube for the passage of food from the pharynx to the stomach; the “food pipe.”

### **Fistula; TE-fistula**

An abnormal duct or passage that connects an abscess or cavity to the body surface; a small opening between the trachea and the esophagus.

### **Indications**

A reason to prescribe a drug or perform a procedure.

### **Tracheoesophageal puncture (TEP)**

An opening made by a surgeon between the trachea and the esophagus. The opening is made at the time of initial surgery or later so that a voice prosthesis can be placed into this passage.

### **Trachea**

The airway through which respiratory air passes to and from the lungs (windpipe).

---

**Tracheoesophageal (TE)**

Pertaining to the trachea and esophagus.

**Tracheostoma**

An opening through the neck and into the trachea. It is through this opening that the laryngectomee breathes.

**Voice prosthesis**

A one-way speech valve that is inserted into the TEP to allow air to pass from the trachea through the prosthesis and into the esophagus to produce sound.

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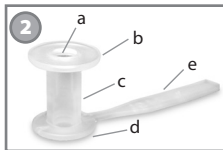
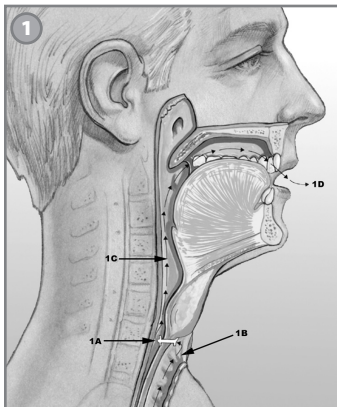
Additional references available upon request.

Weitere Referenzen aus der Literatur sind auf Anfrage erhältlich. / Se pueden solicitar más referencias. / Autres références disponibles sur demande. / Riferimenti aggiuntivi sono disponibili a richiesta. / Meer informatie is op aanvraag beschikbaar. / Referências adicionais mediante solicitação. / Další odkazy na literaturu jsou k dispozici na požádání. / Dodatkowe informacje dostępne na żądanie.

1. Blom, E.D.: "Some Comments on the Escalation of Tracheoesophageal Voice Prosthesis Dimensions," **Arch Otolaryngol**, 129: 500-502, 2003.
2. Eerenstein, S.E., Grolman, W., Schouwenburg, P.F.: "Downsizing of Voice Prosthesis Diameter in Patients with Laryngectomy," **Arch Otolaryngol**, 128: 838-841, 2003.

- 
3. Hamaker, R.C., Blom, E.D.: "Botulinum Neurotoxin for Pharyngeal Constrictor Muscle Spasm in Tracheoesophageal Voice Restoration," **Laryngoscope**, 113:1479-1482, 2003.
  4. Evans, P.H.R., Blom, E.D.: "Functional Restoration of Speech," **Principles and Practice of Head and Neck Oncology**, Evans, P.H.R., Montgomery, P.W., Gullane, P.J. (Eds.) Taylor and Francis Group, London, 2003.
  5. Blom, E.D., Singer M.I., and Hamaker, R.C., **Tracheoesophageal Voice Restoration Following Total Laryngectomy**. San Diego: Singular Publishing Group, Inc.1998.
  6. Blom, E.D., Hamaker, R.C.: "Tracheoesophageal Voice Restoration Following Total Laryngectomy," **Cancer of the Head and Neck**, E.N. and Suen, J. (Eds). W.B. Saunders Publishers. 839-852, 1996.
  7. Blom, E.D., "Tracheoesophageal Speech," **Seminars in Speech and Language**. New York: Thieme Medical Publishers. 16:3, 191-204, 1995.
  8. Pauloski, B.R., Blom, E.D., Logemann, J.A., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part II: Swallow Characteristics," **Laryngoscope**, 105:10: 1104-1110, 1995.
  9. Blom, E.D., Pauloski, B.R., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part I: Speech Characteristics," **Laryngoscope**, 105:10: 1093-1103, 1995.
  10. Blom, E.D., Hamaker, R.C., Freeman, S.B.: "Postlaryngectomy Voice Restoration," Lucente, F.E. (Ed.): **Highlights of the Instructional Courses**, Vol. 7. St. Louis: Mosby-Year Book. pp 3-10, 1994.
  11. Blom, E.D., Singer M.I., and Hamaker, R.C. "A Prospective Study of Tracheoesophageal Speech," **Arch Otolaryngol**, 112: 440-447, 1986.
  12. Blom, E.D., Singer, M.I. "Disinfection of Silicone Voice Prostheses," **Arch Otolaryngol**, 112: 1986.
  13. Blom, E.D., Singer M.I., and Hamaker, R.C. "An Improved Esophageal Insufflation Test," **Arch Otolaryngol**, 111: 211-212, 1985.
  14. Hamaker, R.C., Singer M.I., Blom E.D. and Daniels, H.A. "Primary Voice Restoration at Laryngectomy," **Arch Otolaryngol**, 111: 182-186, 1985.
  15. Singer, M.I. and Blom, E.D. "A Selective Myotomy for Voice Restoration After Total Laryngectomy," **Arch Otolaryngol**, 107: 670-673, 1981.
  16. Singer, M.I. and Blom, E.D. "An Endoscopic Technique for Restoration of Voice After Laryngectomy," **Ann Otol Rhinol Laryngol**, 89:6: 529-533, 1980.

# INHEALTH™

TECHNOLOGIES

---

## UNITED STATES

1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
Toll-Free: 800.477.5969  
Toll-Free Fax: 888.371.1530

## INTERNATIONAL

Tel: +1.805.684.9337  
Fax: +1.805.684.8594  
E-mail: [order@inhealth.com](mailto:order@inhealth.com)

## ORDER ONLINE

[www.inhealth.com](http://www.inhealth.com)

**Freudenberg Medical, LLC**  
1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
[www.inhealth.com](http://www.inhealth.com)

**EMERGO EUROPE**  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands