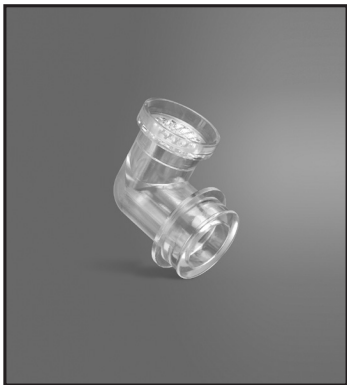
# INHEALTH™

TECHNOLOGIES

## Blom-Singer® Showerguard

USER Instructions For Use

**Blom-Singer®**

voice restoration systems

**R1**

37728-01E

37728-01E Effective January 2016 / Gültig ab Januar 2016 /  
Efectivo a partir de enero de 2016 / Date d'entrée en vigueur :  
janvier 2016 / Valido da gennaio 2016 / Met ingang van januari  
2016 / Vigente em janeiro de 2016 / V platnosti od ledna 2016 /  
Wchodzi w życie w styczniu 2016 r.

**Blom-Singer** and **InHealth Technologies** are registered  
trademarks of **Freudenberg Medical, LLC**

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### **BLOM-SINGER® SHOWERGUARD**

The Blom-Singer Showerguard is designed to shield the tracheostoma when showering and washing your hair. When properly applied it will permit you to breathe freely while preventing the shower spray, and water that runs down the neck, from entering the tracheostoma.

### **INDICATIONS**

The Blom-Singer Showerguard is designed for use by individuals who have a permanent opening in their neck (stoma) following removal of the voice box (laryngectomy).

### **CONTRAINDICATIONS**

The Blom-Singer Showerguard is designed only for use in protecting the airway opening (stoma) during showering and must be properly attached as described below. This device is not designed to protect the airway if the head and neck are submerged in water. It must not be used in any water sports.

### **HOW SUPPLIED**

The Blom-Singer Showerguard is supplied nonsterile in a single wrap packaging system and is available in one universal size only.

### **WARNINGS AND PRECAUTIONS**

The Blom-Singer Showerguard is designed only for use in protecting the airway opening (tracheostoma) during showering and must be properly attached as described below.

The Showerguard is not designed to protect the airway if the head and neck are submerged in water.

The Showerguard must not be used in any water sports.

If there are tears, cracks, or structural damage to the device, discontinue use.

**Caution:** Federal (USA) law restricts this device to sale by or on the order of a physician or a licensed speech pathologist.

---

## COMPLICATIONS

The penetration of a small amount of shower spray or droplets of water into the airway (stoma) causing coughing. This indicates absence of a water-tight seal and the user should immediately step out of the shower and reposition the device.

## INSTRUCTIONS FOR USE

### **Insertion, Removal and Cleaning**

A properly attached Showerguard system provides an airtight seal around the stoma. Before inserting the Showerguard, a Blom-Singer tracheostoma housing must first be attached to the skin surrounding the tracheostoma. Follow the attachment steps described in the Blom-Singer Tracheostoma Housings Instructions For Use provided with the product.

#### **Insertion**

To insert the Showerguard, position it with the perforated breathing filter facing downward (diagram 1) and insert it partially into the lower rim of the previously applied tracheostoma housing. Gently push the Showerguard until it snaps into place. A properly inserted Showerguard will fit squarely inside the housing.

#### **Removal**

The Showerguard is designed for instant removal without removing the tracheostoma housing from the skin. Place your index finger firmly against the housing to prevent pulling it loose. Grasp the Showerguard by the outer rim and pull to remove.

#### **Cleaning**

The tubular vent (diagram 2a) and the perforated breathing filter (diagram 2b) can be taken apart for cleaning and easily reassembled.

Rinse with clean water and either air-dry or use a lint-free material to dry the device. Never use facial or toilet tissue to dry the Showerguard as particles of lint or fabric may be deposited onto the device that can be inhaled into the airway upon reinsertion.

Once a Showerguard has been cleaned and dried completely, it should be stored in a clean, reclosable plastic bag until its next use.

## **SPECIAL ORDER PRODUCTS**

If this instruction manual accompanies a Special Order Product, there may be differences in the physical characteristics between the product and the product descriptions described in this instruction manual. These differences will not affect the safety or desired beneficial effect of the special order product. Special Order products are nonreturnable.

## **ORDERING INFORMATION**

### **USA**

Blom-Singer products may be ordered directly from InHealth Technologies. TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337, Monday–Friday, 9:30am–7:00pm, Eastern Standard Time. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: [order@inhealth.com](mailto:order@inhealth.com) ORDER ON-LINE: [www.inhealth.com](http://www.inhealth.com) POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service

### **International**

Please contact our customer service department for a distributor referral.

### **Consumer Affairs**

If you have any questions or dissatisfaction with a product please contact our customer service department via telephone, fax, post, or Email: [productcomplaints@inhealth.com](mailto:productcomplaints@inhealth.com)

## **GLOSSARY**

### **Airway**

The air passage by which air reaches the lungs.

### **Candida albicans**

A yeast fungus that is a commonly occurring condition that can infect the mouth or the skin and may multiply and overgrow on to the voice prosthesis.

### **Clinician**

A health professional, such as a physician, licensed speech-language pathologist, or nurse who is involved in clinical practice.

---

### **Contraindications**

A reason that makes it inadvisable to prescribe a particular procedure or treatment.

### **Esophagus**

The muscular membranous tube for the passage of food from the pharynx to the stomach; the “food pipe.”

### **Fistula; TE-fistula**

An abnormal duct or passage that connects an abscess or cavity to the body surface; a small opening between the trachea and the esophagus.

### **Indications**

A reason to prescribe a drug or perform a procedure.

### **Occlusion (Occlude)**

To block, such as the stoma with a finger, thumb, or ATSV in order to produce tracheoesophageal voice.

### **Respiration**

The act or process of inhaling and exhaling; breathing; moving air into and out of the lungs. Also called ventilation.

### **Respiratory Tract (laryngectomee)**

The air passage from the tracheostoma to the air sacs of the lungs, including the trachea, and bronchi through which air passes during breathing.

### **Tracheoesophageal puncture (TEP)**

An opening made by a surgeon between the trachea and the esophagus. The opening is made at the time of initial surgery or later so that a voice prosthesis can be placed into this passage.

### **Trachea**

The airway through which respiratory air passes to and from the lungs (windpipe).

### **Tracheoesophageal (TE)**

Pertaining to the trachea and esophagus.

**Tracheostoma**

An opening through the neck and into the trachea. It is through this opening that the laryngectomee breathes.

**Voice prosthesis**

A one-way speech valve that is inserted into the TEP to allow air to pass from the trachea through the prosthesis and into the esophagus to produce sound.

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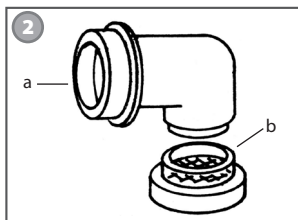
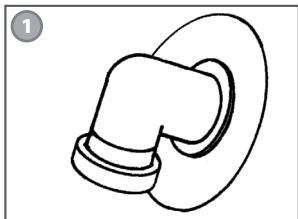
Additional references available upon request.

Weitere Referenzen aus der Literatur sind auf Anfrage erhältlich. / Se pueden solicitar más referencias. / Autres références disponibles sur demande. / Riferimenti aggiuntivi sono disponibili a richiesta. / Meer informatie is op aanvraag beschikbaar. / Referências adicionais mediante solicitação. / Další odkazy na literaturu jsou k dispozici na požádání. / Dodatkowe informacje dostępne na żądanie.

1. Evans, P.H.R., Blom, E.D.: "Functional Restoration of Speech," **Principles and Practice of Head and Neck Oncology**, Evans, P.H.R., Montgomery, P.W., Gullane, P.J. (Eds.) Taylor and Francis Group, London, 2003.
2. Blom, E.D., Singer M.I., and Hamaker, R.C., **Tracheoesophageal Voice Restoration Following Total Laryngectomy**. San Diego: Singular Publishing Group, Inc.1998.
3. Blom, E.D., Hamaker, R.C.: "Tracheoesophageal Voice Restoration Following Total Laryngectomy," **Cancer of the Head and Neck**, E.N. and Suen, J. (Eds). W.B. Saunders Publishers. 839-852, 1996.
4. Blom, E.D., "Tracheoesophageal Speech," **Seminars in Speech and Language**. New York: Thieme Medical Publishers. 16:3, 191-204, 1995.
5. Blom, E.D., Hamaker, R.C., Freeman, S.B.: "Postlaryngectomy Voice Restoration," Lucente, F.E. (Ed.): **Highlights of the Instructional Courses**, Vol. 7. St. Louis: Mosby-Year Book. pp 3-10, 1994.

# INHEALTH™

TECHNOLOGIES

---

## UNITED STATES

1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
Toll-Free: 800.477.5969  
Toll-Free Fax: 888.371.1530

## INTERNATIONAL

Tel: +1.805.684.9337  
Fax: +1.805.684.8594  
E-mail: [order@inhealth.com](mailto:order@inhealth.com)

## ORDER ONLINE

[www.inhealth.com](http://www.inhealth.com)

**Freudenberg Medical, LLC**  
1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
[www.inhealth.com](http://www.inhealth.com)

**EMERGO EUROPE**  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands